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🏢 Ningbo Caremet Medical Products Co., Ltd.
Building 1#5-4, 36 Qiangfeng Road, Wanyang Industrial Zone, Beilun Dist.,
Ningbo 315813, Zhejiang Province, China

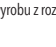
🇪🇺 SUNGO Europe B.V.
Fascinatio Boulevard 522, Unit 1.7, 2909VA Capelle aan den IJssel

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PRODUCT INFORMATION

Elastic cohesive bandage. Model: KMT101, KMT103

INTENDED USE:

Bandage is intended to act as secondary dressings to provide compression control. This is a single use, disposable device(s), provided non-sterile.

STORAGE:

The product should be stored in normal temperature and humidity no more than 80%, away from heat and direct sunlight.

TRANSPORTATION:

The product should be protected from rain and moisture during transportation.

INSTRUCTION FOR USE

Remove the package of the bandage, wind the bandage around the part which needs to be fixed, cut it in a proper length, overlap the end of the bandage (about 5cm length at the end to the one which have been already wind, press the overlapping part.

Shelf Life: 3 years

CAUTIONS

- △ The product must be used on intact (uninjured) skin only. Wounds must be treated with a suitable wound dressing in advance.
- △ Single use, do not reused.
- △ This product has certain elasticity, it require appropriate pressure and parcel number layers when binding or fixing, not too tight or too loose, too tight will discomfort or adverse reactions to the patient which will cause partial blood-supply such as local blood supply bring patients, too loose will not keep wound dressing fixed effectively.

DISPOSAL

Please dispose the product after use to comply with local regulation.

CONTACT INFORMATION

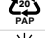
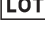
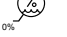
 Ningbo Caremet Medical Products Co., Ltd.
 Building 1#5-4, 36 Qiangfeng Road, Wanyang Industrial Zone, Beilun Dist.,
 Ningbo 315813, Zhejiang Province, China

 SUNGO Europe B.V.
 Fascinatio Boulevard 522, Unit 1.7, 2909VA Capelle aan den IJssel

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Made in P.R.C.

SYMBOLS USED

SYMBOL	MEANING	SYMBOL	MEANING
	Non-sterile product		Read the instructions manual
	Single-use only		Medical device
	Do not use if the packaging was damaged		Keep away from moisture
	Paper part of the packaging is recyclable		Contains latex
	Keep away from direct sunlight		Authorised European Union representative
	Manufacturer		Manufacturing date
	Lot number		Indicates the range of humidity to which the medical device can be safely exposed
	The CE marking symbolizes the product's compliance with Regulation (EU) 2017/745 of the European Parliament and of the Council.		

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Novamed.pl S.A.